

omniCHROMA®

Product Family

LOOKBOOK 4





“It is said that matching the color of a single tooth is the hardest thing we do as dentists. Actually, matching a portion of a single tooth is even harder. **OMNICHROMA** takes the guess work out of picking shades for aesthetic composite restorations. Finally...one shade truly matches all.”

-Dr. Robert Lowe, Charlotte, NC

Table of Contents

ANTERIOR TECHNIQUE	5-7
POSTERIOR TECHNIQUE	9-11
WHITENING	13
VENEERS	14
DIASTEMA	15
BLACK TRIANGLE	16
COSMETIC BONDING	17
CLASS III	19-21
CLASS IV	22-24
CLASS V	25-27
CLASS I	29-31
CLASS II	32-34



“ I specialize in polychromatic layering of many shades of composite to achieve the most natural results possible. When I was first introduced to **OMNICHROMA**, I had serious doubts about its ability to live up to the hype. I questioned how a single shade composite system could even come close to a layered technique.

*I now use **OMNICHROMA** products for all my posterior direct restorations, and over half of the esthetically challenging anterior direct restorations.”*

-Dr. Kevin Brown, Bellevue, WA

30-year-old patient presented with **Class IV** fracture of **#8** Mesial-Incisal, **shade A2/A3**.
OMNICHROMA and **OMNICHROMA BLOCKER** were used with the aid of a Mylar strip to create a palatal shell.



Pre-operative



Tooth prepped



Palatal shell created with a thin layer of
OMNICHROMA

Courtesy of Dr. Kevin Brown, Bellevue, WA



OMNICHROMA BLOCKER was layered sufficient to block out the hard incisal edge



A thin layer of OMNICHROMA was applied and cured



Colored resin applied with a sable brush to create characterization

Courtesy of Dr. Kevin Brown, Bellevue, WA



Final layer of **OMNICHROMA**
was placed and cured



Polished with fine grit diamonds,
discs, and complete polishers



Final restoration

Courtesy of Dr. Kevin Brown, Bellevue, WA



“ From day 1, I was so impressed and satisfied with this new Smart Chromatic Technology by Tokuyama that now, whenever I need composite, I use **OMNICHROMA**. This composite’s consistency handles very well. Its moderate stiffness makes it easy to manipulate, contour, polish, and shine.

With **OMNICHROMA**, you can obtain highly esthetic results fast, easily, and predictably. It cannot be simpler as there’s only 1 shade. There’s no waste of time in shade taking, tracking, and recording all colors used for 1 restoration, and keeping up with your inventory.”

-Dr. Juan Uribe, Sioux Center, IA

25-year-old male presented with multiple caries, **shade A2/A3**. **OMNICHROMA BLOCKER Flow**, **OMNICHROMA Flow BULK**, and **OMNICHROMA** were used on **Teeth #2-5**.



Pre-operative



Teeth #2-5, prepped



Resin-modified calcium silicate filled liner was applied, followed by Tokuyama Universal Bond on Tooth #3.

Courtesy of Dr. Juan Uribe, Sioux Center, IA



OMNICHROMA *Flow Bulk* applied to teeth #2 and #3



OMNICHROMA applied to teeth #2 and #3



Teeth #4 and #5 with OMNICHROMA *BLOCKER Flow*.

Courtesy of Dr. Juan Uribe, Sioux Center, IA



Composite cured



Final restoration



Bite checked

Courtesy of Dr. Juan Uribe, Sioux Center, IA



“**OMNICHROMA** [products] have changed my practice flow for the better! A lot of my patients are extremely esthetically driven, and I **NEED** to perfectly match their existing tooth shade to make them happy. This is so much easier with **OMNICHROMA** as it matches any shade.

Another **MAJOR** advantage is that it whitens with the tooth due to its shade matching technology. I could not ask for an easier and more beautiful product to use!”

-Dr. Lisel Balk, Delray Beach, FL



Case Study Article:
**Read more about
this whitening case**



Patient consistently used OTC bleaching agents for 3 years resulting in a mismatched Class IV on upper left central incisor, #9. **OMNICHROMA BLOCKER & OMNICHROMA** were used to replace the old restoration. **Twenty-four months** and many additional bleaching strips later, **OMNICHROMA** continues to shade match.

Courtesy of Dr. Robert S. Rosenfeld, Westwood, NJ

A Word from Dr. Nash:

"I use [OMNICHROMA products] in my practice almost every day to provide direct esthetic restorations, for repairs of composite and porcelain, and for mock ups for patients who want to envision the result before embarking upon treatment. Thank you for such a useful product."



17-year-old who had unusually narrow maxillary incisors and spaces left after orthodontic treatment, **shade A1. OMNICHROMA BLOCKER** was used in the proximal areas followed by **OMNICHROMA** for the facial veneers on **Teeth #6-11.**

Courtesy of Dr. Ross Nash, Huntersville, NC



A 55-year-old female has had a small diastema between **#8 & 9 (B1)** for many years. She complained that others would mention that she had “pepper between her teeth.” We closed the space with no preparation using **OMNICHROMA Flow** as well as some **OMNICHROMA** to complete the final shape.

Courtesy of Dr. Mike Malone, Lafayette, LA



Patient presented with a black triangle between **Teeth #8-9**. Using **OMNICHROMA Flow**, we added to the mesial of both teeth to close the diastema.

Courtesy of Dr. Lisel Balk, Delray Beach, FL

A Word from Dr. Rosenfeld:

"OMNICHROMA has proven its ability to match shades from light to dark. A significant percentage of my patient population are seniors, with teeth darker than the standard A2. OMNICHROMA has eliminated the shade-matching step, and has eliminated the need to blend shades to try and duplicate Mother Nature."



Patient presented with a crack in **Tooth #9**, shade about **A3.5** or **D3**, and wanted it fixed.

A shallow prep was made on the gingivo-incisal length of the tooth
and restored with **OMNICHROMA Flow**.

Courtesy of Dr. Robert S. Rosenfeld, Westwood, NJ



“ I was pretty amazed that **OMNICHROMA** was able to not only match the shade of the surrounding tooth structure, but also mimic some of the opalescence found at the incisal edge of the tooth.

This product works well for treating incisal edge fractures.”

-Dr. Nathaniel Lawson, Birmingham, AL



Patient presented with **Class III, shade A2.**
Restored with **OMNICHROMA.**

Courtesy of Dr. Peter Auster, Pomona, NY



Tooth #7 (DF & MFL) and **#8** (MFL). Direct composite restoration completed with **OMNICHROMA**. Enamel was beveled and striations in the enamel created to better blend the composite with the natural tooth.

Courtesy of Dr. Neville Hatfield, Hoboken, NJ



Interproximal decay removed on **Teeth #2-4, shade A2.**
OMNICHROMA Flow BULK was used.

Courtesy of Dr. Kevin Brown, Bellevue, WA



Patient presented with a chip on incisal edge of **Tooth #9**. A bevel was placed on the enamel with a diamond rotary. A finger was placed on the lingual surface of the tooth and a thin increment of **OMNICHROMA** was placed and light cured. A second increment of **OMNICHROMA** was placed and adapted. After light curing, the restoration was finished with a rubber finishing point and polished with a spiral-shaped polisher.

Courtesy of Dr. Nathaniel Lawson, Birmingham, AL



20-year-old presented as an emergency walk-in patient. **Tooth #9** fractured and patient was asymptomatic and most concerned about esthetics. Used **OMNICHROMA BLOCKER & OMNICHROMA** composites.

Courtesy of Dr. Kushbu Shah, Chicago, IL



A 56-year-old female patient complained of a chipped upper right central incisor (**Tooth #8**).

The material selected for this procedure was
OMNICHROMA BLOCKER *Flow* and **OMNICHROMA *Flow***.

Courtesy of Dr. James Peyton, Bakersfield, CA



10-year-old child, **shade A2/A3**, with congenital hypocalcification and poor hygiene which progressed into decay. Decay needed to be repaired prior to starting orthodontics.

OMNICHROMA was used for this restoration.

Courtesy of Dr. Kevin Brown, Bellevue, WA



High caries risk male. Excavated caries under careful isolation, **Tooth #4** Distal-Facial, and restored with **OMNICHROMA**.

Courtesy of Dr. Melissa Seibert, San Angelo, TX



A 55-year-old male wanted to repair root resorption on his lower left first bicuspid (#20, Shade A3). **OMNICHROMA Flow** was used.

Courtesy of Dr. Mike Malone, Lafayette, LA



“**OMNICHROMA BLOCKER Flow** is a must have in every composite arsenal. It effortlessly covers dark dentin and discoloration. **OMNICHROMA BLOCKER Flow**, with the original **OMNICHROMA** or **OMNICHROMA Flow**, makes restorations very simple.”

-Dr. Ian Shuman, Glen Burnie, MD

A Word from Dr. Chae:
*"I always recommend a nice bevel
because it allows for a more seamless
and natural looking blend."*



Bitewing x-ray showed very deep caries close to the pulp on **Tooth #19, shade A2**.
Placed pulp capping liner, then **OMNICHROMA Flow BULK**.

Courtesy of Dr. James Chae, Diamond Bar, CA



Recurrent caries under the old amalgam restoration on

Tooth #19, shade A3.5.

OMNICHROMA Flow BULK was used.

Courtesy of Dr. James Chae, Diamond Bar, CA



Recurrent caries in the pits and grooves under the old sealant on **Tooth #31, shade A2**.
OMNICHROMA Flow was used.

Courtesy of Dr. James Chae, Diamond Bar, CA



Procedure Video:

Watch Dr. Brown replace
an amalgam restoration



Replacing old amalgam restoration on **Teeth #12-14, shade A1/B1**. Used **OMNICHROMA BLOCKER Flow** to mask staining, then placed **OMNICHROMA Flow BULK** on top.

Courtesy of Dr. Kevin Brown, Bellevue, WA



Simple occlusal decay on **Teeth #30-31, shade A2/A3.**
OMNICHROMA Flow BULK was used.

Courtesy of Dr. Kevin Brown, Bellevue, WA



Small interproximal caries on **Teeth #12-14**.
OMNICHROMA Flow was used.

Courtesy of Dr. Kevin Brown, Bellevue, WA



OMNICHROMA Flow
Category: New Products



OMNICHROMA Flow BULK
Category: New Products



EVERY SHADE. ONE CHOICE.

With its award-winning technology and dependable results, OMNICHROMA products are a one-shade composite system that you can rely on.

Tokuyama Dental America would like to thank the doctors who have contributed their case images and testimonials to this Lookbook.

Thank you for trusting our products.



Innovating Tomorrow's Dentistry, Today.™