

LOOKBOOK 5



OmniCHROMA

ONE SHADE UNIVERSAL COMPOSITES

"It is said that matching the color of a single tooth is the hardest thing we do as dentists. Actually, matching a portion of a single tooth is even harder. **OMNICHROMA** takes the guess work out of picking shades for aesthetic composite restorations.

Finally... one shade truly matches all."

Dr. Robert Lowe
Charlotte, NC

With award-winning technology and dependable results,
OMNICHROMA Product(s) are a one-shade composite
system you can rely on!

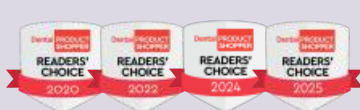


Table of Contents

Technique: Class IV	4-5
Technique: Full Mouth Restoration	6-7
Class I	8-9
Class II	10-12
Class III	13-14
Class IV	15-18
Class V	19-21
Cracked Tooth and Calcification	22-24
Endo, Screw Holes	25-26
Esthetic	27-31
Longevity	32-37

Technique: Class IV

Teeth: **#8-10** | Shade: **C2**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER**

70+ year-old male with worn incisal edges.



Pre-operative smile



Sand blasted



0.5mm lingual shelf with
OMNICHROMA BLOCKER



OMNICHROMA on the facial



Immediate post-operative



Post-operative



"I questioned how a single shade composite system could even come close to a layered technique. I now use OMNICHROMA products for all my posterior direct restorations, and over half of the esthetically challenging anterior direct restorations."

Dr. Kevin Brown, Bellevue, WA

Technique: Full Mouth Restoration

Class: **I, II, III, IV** | Teeth: **#2-5, 7, 9-10, 29, 31** | Shade: **A2/A3**
Product(s): **OMNICHROMA, OMNICHROMA BLOCKER, OMNICHROMA BLOCKER Flow, OMNICHROMA Flow BULK**

25-year-old male with multiple caries, root remain on #8.



Pre-operative



Teeth 7 and 9 prepped and bonded



OMNICHROMA BLOCKER
applied on lingual shelf



OMNICHROMA applied on facial



Finish and polish

Technique: Full Mouth Restoration



Pre-operative



Teeth #2-5 prepped



#3 with **OMNICHROMA BLOCKER**



#3 with **OMNICHROMA Flow BULK**



#2, 4-5 with **OMNICHROMA**



Post-operative



"Today, I am using OMNICHROMA for 90% of my cases, both simple and complex. It handles beautifully, stays put, and you can shape it easily."

Dr. Juan Uribe, Sioux Center, IA

Class I

Tooth: **#19** | Shade: **A3.5**
Product(s): **OMNICHROMA Flow BULK**

35-year-old male patient with recurrent caries under old amalgam.



Pre-operative



Tooth prepped



Post-operative



"I have been using dental products from Tokuyama Dental for more than 13 years. After trying OMNICHROMA Flow BULK, I was amazed how well this one-shade flowable bulk fill blended in and matched the shade of the tooth. It is very simple to use and easy to polish."

Tooth: **#30** | Shade: **A1**
Product(s): **OMNICHROMA Flow BULK**

New decay was detected. Restored without the need for anesthesia.



Pre-operative



Post-operative

Dr. Eric Block, Acton, MA

Class II

Tooth: **#12-14** | Shade: **A1/B1**

Product(s): **OMNICHROMA BLOCKER Flow, OMNICHROMA Flow BULK**

40-year-old patient with old amalgam and heavy staining.



Pre-operative



OMNICHROMA BLOCKER Flow, about 1mm thick



Single layer of **OMNICHROMA Flow BULK**, about 2mm thick



Post-operative

Teeth: **#3-5** | Shade: **A3/A2**
Product(s): **OMNICHROMA Flow**

Small interproximal caries restored with very conservative slot preps.



Pre-operative



Teeth prepped



Post-operative

Class II

Tooth: **#3** | Shade: **A2**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER Flow**

Patient with a cracked filling on her mesiobuccal cusp and recurrent decay.



Pre-operative



Post-operative

Tooth: **#7** | Shade: **A3**
Product(s): **OMNICHROMA, OMNICHROMA BLOCKER,
Tokuyama Universal Bond**

76-year-old male with recurrent caries and leaking restoration.



Pre-operative



Post-operative



"With OMNICHROMA, you can obtain highly esthetic results fast, easily, and predictably. There's no waste of time in shade taking, tracking, and recording all colors used for 1 restoration, and keeping up with your inventory."

Dr. Juan Uribe, Sioux Center, IA

Class III & Class V

Teeth: **#6-11** | Shade: **C2/A3.5**

Product(s): **OMNICHROMA BLOCKER Flow, OMNICHROMA Flow**

23-year-old female with chronic facial and interproximal decay.



Pre-operative



Post-operative

Tooth: **#9** | Shade: **A2**

Product(s): **Estelite OMEGA, OMNICHROMA Flow**

32-year-old male with cracked tooth repaired. Lingual layer built using **Estelite OMEGA DA2**, then a layer of **DA1** to mimic dentin and enamel. **OMNICHROMA Flow** was applied as the final layer.



Pre-operative



Post-operative

Class IV

Teeth: **#13-23, #33-43** | Shade: **A2 to OM3 post-whitening**

Product(s): **OMNICHROMA, OMNICHROMA Flow**

After aligner and whitening treatments, patient requested squarer incisal embrasures.



Pre-operative



Post-operative

Tooth: **#9** | Shade: **B1**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER Flow**

Patient presented with incisal fracture and brought in incomplete fragment. Without anesthesia, we sandblasted both the fragment and tooth. Using **OMNICHROMA BLOCKER Flow** as an initial "cement," we were able to eliminate any light shine-through and filled in the remaining defects with **OMNICHROMA**.



Pre-operative



Tooth fragment cemented with
OMNICHROMA BLOCKER Flow



Post-operative

Tooth: **#8** | Shade: **A1/A2**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER**

Patient with broken incisal edge with pre-existing composite.



Prepped tooth



Lingual shelf with **OMNICHROMA
BLOCKER** applied



Post-operative

Tooth: **#9** | Shade: **C2/A3.5**
Product(s): **OMNICHROMA, OMNICHROMA BLOCKER,
Tokuyama Universal Bond**

Patient with disto-buccal decay on #9 has a dark gingival shade and a more translucent interproximal shade.



Pre-operative



Post-operative



***"Note that OMNICHROMA blended well with
both shades at the same time!"***

Dr. Peter Auster, Pomona, NY

Class V

Tooth: **#11** | Shade: **B3**
Product(s): **OMNICHROMA Flow**

51-year-old patient with a chief complaint that she did not like the decay and stain at the gum line.



Pre-operative



Post-operative

Teeth: **#3-5** | Shade: **A4 at Gingival**
Product(s): **OMNICHROMA BLOCKER Flow, OMNICHROMA Flow**


Abfraction and asymmetric smile.



Pre-operative



Post-operative

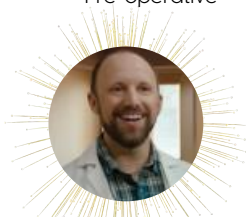

Dr. Chris Nelson, Redding, CA
Winner of the 2024 Case Image Contest!



Cracked Tooth

Tooth: **#9** | Shade: **A3.5/D3**
Product(s): **OMNICHROMA Flow**

A senior patient presented with a crack in tooth #9. A shallow prep was made on the gingivo-incisal length of the tooth and restored.



Pre-operative



OMNICHROMA Flow applied



Post-operative



"OMNICHROMA Flow eliminates the complexity of trying to select appropriate shades, allows excellent adaptation to cavity walls, and the addition of exceptionally thin, uniform layers of materials when desirable... All with a single-shade of composite."

Dr. Robert Rosenfeld, Westwood, NJ

Tooth: **#10** | Shade: **A2**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER**

Patient with large class IV bonding failure with hypocalcification and fluorosis.



Pre-operative



Post-operative

Calcification

Tooth: **#8** | Shade: **A1**

Product(s): **OMNICHROMA, Tokuyama Universal Bond**

Juvenile patient thought she needed porcelain veneers after severe decalcification following orthodontics. Gave them several options and they opted for a bonded restoration.



Pre-operative



Post-operative



"Note excellent result using OMNICHROMA to replace damaged enamel from decalcification!"

Dr. Peter Auster, Pomona, NY

Tooth: **#2** | Shade: **No shade taken, possibly C2/C3**
Product(s): **OMNICHROMA BLOCKER, Tokuyama Universal Bond**

Patient presented with an endo access hole in a ceramic crown.



Pre-operative



Post-operative

Screw Holes

Tooth: **#18** | Shade: **A2**

Product(s): **OMNICHROMA BLOCKER Flow, OMNICHROMA Flow**

Gray screw hole covered with teflon tape, cotton pellet, and **OMNICHROMA BLOCKER Flow** & **OMNICHROMA Flow**.



Pre-operative



Post-operative

Teeth: **#6-11** | Shade: **A1**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER**

16-year-old, orthodontic treatment with unusually narrow maxillary anterior teeth so spaces existed after.



Pre-operative



Post-operative



"I use [OMNICHROMA products] in my practice almost every day to provide direct esthetic restorations, for repairs of composite and porcelain, and for mock ups for patients who want to envision the result before embarking upon treatment."

Dr. Ross Nash, Huntersville, NC

Pediatric Strip Crowns

Teeth : **#D, E, F, G** | Shade: **A1**
Product(s): **OMNICHROMA Flow BULK**

Patient with early childhood caries in all primary maxillary incisors. Strip crowns, size 3, were used, with **OMNICHROMA Flow BULK**.



Pre-operative



Post-operative

Teeth: **#D, E** | Shade: **A1**
Product(s): **OMNICHROMA Flow**

4-year-old presented with caries. Teeth were prepared for a class V restoration on #E and a class IV on #D.



Pre-operative



Immediate post-operative

Diastema & Class IV

Teeth: **#8-9** | Shade: **A1**
Product(s): **OMNICHROMA**

40-year-old female presented with concern about a small diastema between central incisors, and mismatched composite on #9.



Pre-operative



Post-operative

Teeth: **#24-25** | Shade: **D2/D3**

Product(s): **OMNICHROMA, OMNICHROMA BLOCKER, OMNICHROMA BLOCKER Flow**

OMNICHROMA BLOCKER Flow was sculpted along the cervical third, followed by an initial layer of **OMNICHROMA** placed on facial and cervical thirds. Using a curved mylar, a final layer of **OMNICHROMA** was placed on the facial and **OMNICHROMA Flow** on the lingual.



Pre-operative



Post-operative

3-Year Follow-Up

Tooth: **#31** | Shade: **A3**
Product(s): **OMNICHROMA Flow BULK**

The **failing sealant** with recurrent caries was removed and the tooth was restored.



Pre-operative



Post-operative



3-Year follow-up



"I always recommend a nice bevel because it allows for a more seamless and natural looking blend."

Dr. James Chae, Diamond Bar, CA

Tooth: **#31** | Shade: **A3**

Product(s): **OMNICHROMA Flow, OMNICHROMA Flow BLOCKER**

Failing amalgam with recurrent caries.



Pre-operative



Immediate post-operative



3-Year follow-up

5-Year Follow-Up

Teeth: **#6-11** | Shade: **A1/A2**

Product(s): **OMNICHROMA and OMNICHROMA BLOCKER #7-10,
OMNICHROMA only on #6 and #11**

Patient presented post-orthodontic treatment with an **old class IV** restoration on #9 and residual spacing after braces.



Pre-operative



Immediate post-operative



5-year follow-up

Teeth: **#24-25** | Shade: **A1**
Product(s) : **OMNICHROMA**

The patient wanted to **close the gap** between the two lower central teeth. No anesthesia was used.



Pre-operative



Immediate post-operative



5-year follow-up

18-Month Follow-Up

Tooth: **Premolars and Molars**

Product(s): **OMNICHROMA**

18 months following **OMNICHROMA** placement – with subsequent tooth whitening – multiple restorations placed in three quadrants still appear new.



18-month follow-up



"[Years later], the material blends in beautifully and looks just as good as the day it was placed!"

Dr. Peter Auster, Pomona, NY

43-Month Follow-Up

Tooth: **#9** | Shade: **A1**
Product(s): **OMNICHROMA**

Patient is a chronic OTC bleacher and presented with dissatisfaction of old composite's mismatching shades.



Pre-operative



Tooth #9 repaired with **OMNICHROMA**



Immediate follow-up



11-month follow-up



23.5-month follow-up



43-month follow-up

Resources

YouTube Videos



Video Testimonial
with Dr. Peter Auster



From the Desk of
Dr. David Chan



BLOCKER Playlist with
Dr. Kevin Brown

Articles



Whitening



Quadrant Dentistry



Screw Holes



Injection Molding

Lookbooks



Lookbook 1



Lookbook 2



Lookbook 3



Lookbook 4

**Tokuyama Dental America would like to thank the
doctors who have contributed their case images
and testimonials to this Lookbook.**

Thank you for trusting our products.

Like these case images and testimonials?

Want to submit your own?





760-942-7211



info@tokuyama-us.com



omnichroma.com/us



Encinitas, CA



Tokuyama



Innovating Tomorrow's Dentistry, Today.™